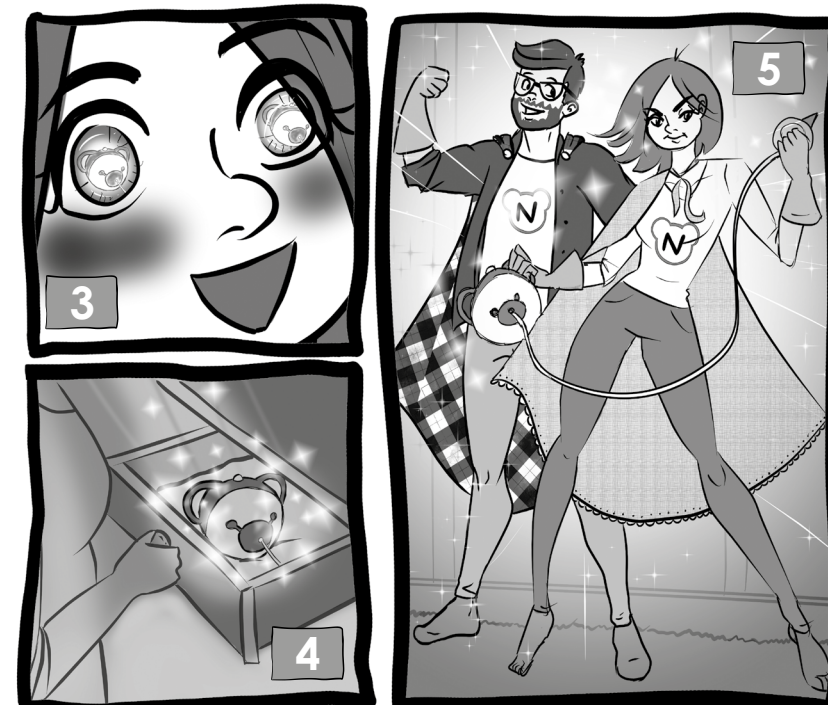
Before using the Nosiboo Pro, carefully read this guide and the Instructions for Use as well! Please keep this guide for future reference.

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
to be continued...

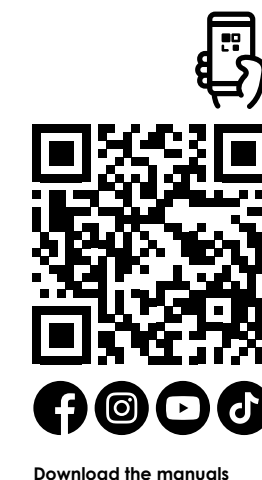Download the manuals

AFTER-SALE SERVICE
For more information about usage, cleaning, and for troubleshooting, see the Instructions for Use.

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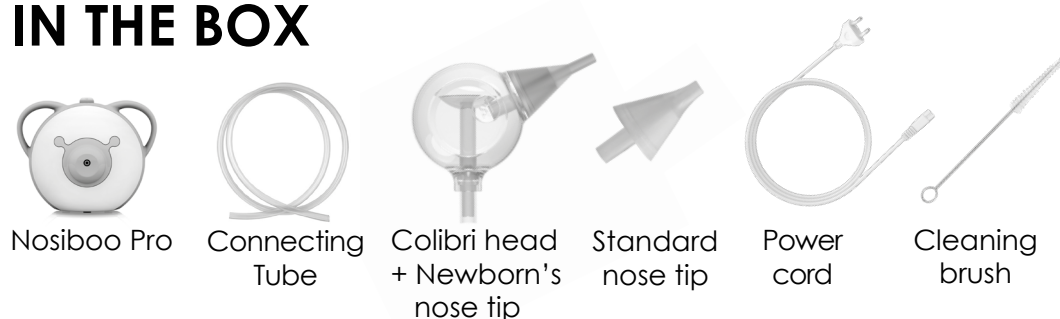
www.nosiboo.com
customerservice@nosiboousa.com
(312) 957-6046

IN THE BOX

Contact the distributor if any part is missing from the package.

USING THE NOSIBOO PRO

The Nosiboo Pro electric nasal aspirator is a medical device intended for use by laypersons. It is designed for use at home to remove nasal mucus from children primarily under the age of 6.

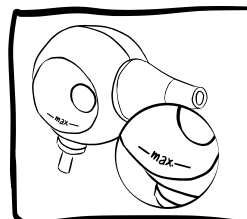
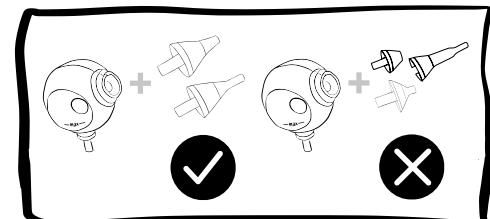
The use of non-original spare parts poses technical and medical risk for which the manufacturer accepts no responsibility! The use of non-original spare parts may cause malfunction or damage to the device and will result in loss of warranty. Only use official Nosiboo parts!


When the Colibri head fills to the marked level (max.), empty it to prevent the secretion from entering the tube.

Aspirating water with the electric device may cause malfunction or damage and will result in loss of warranty.

Before the first use





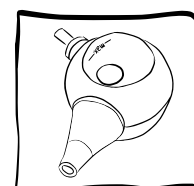
First disassemble the Colibri head and clean and dry its parts.


Check the "CLEANING" section below for more details about the proper cleaning method.

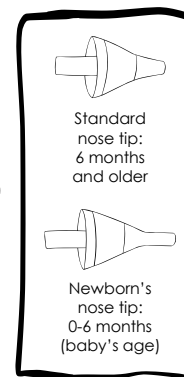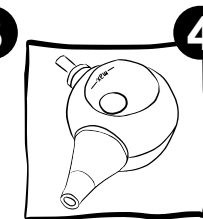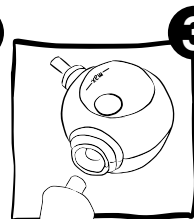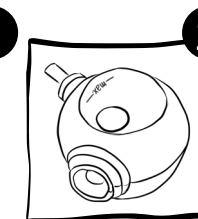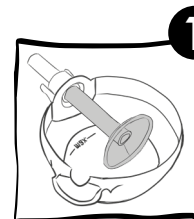Assemble the Colibri head



Check whether the Colibri head is correctly assembled, and the collared tube has not been missed.

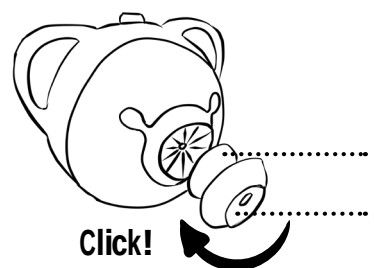
Assemble the remaining parts

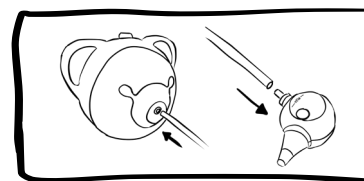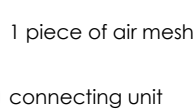Click!
When assembling, turn until it clicks into place.

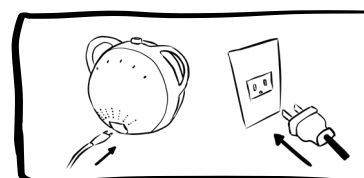
Attach the connecting tube to the device and to the Colibri head.


Attach the power cord to the device and the power outlet. The device is ready to use.

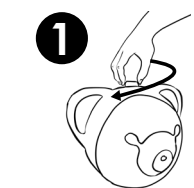INSTRUCTIONS FOR USE & SETTINGS

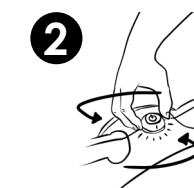Switch on the device by turning the control button.


Select the correct suction power.
Setting recommendations:
• min setting for newborns
• middle setting for babies
• max setting for small children and for pre-schoolers

Hold your child in a stable position.
• Aspirate the nose of infants after laying them into a stable position and hold their head during the process.
• You may also sit them in an infant bouncer or infant seat.
• If the child is already able to sit, sit them in your lap. In this case, do not lay the child on their back during the aspiration.

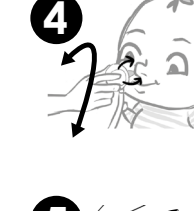
The effective and strong (max) suction power has been proven safe for nasal aspiration by ENT doctors.
If the mucus is thin, you can use the device at a lower setting. If it is thick, you can use the device at a higher setting.


Gently insert the clean, dry nose tip into the child's nostril.
• Move the nose tip around gently to make sure that you get all areas of the nostril.

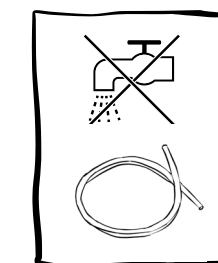
Switch off the device after use with the control button (off setting).
The device should be connected to or disconnected from the power network only if the control button is in off position (the device is switched off).
If you do not use the device for a longer period of time, disconnect it from the power network.

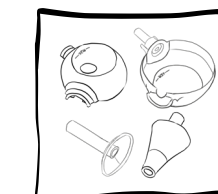
• Don't wash the connecting tube. Next to proper use, the mucus can't enter the tube so washing the tube is not necessary.
• However, if there is any mucus or liquid in the tube, then stop using the device. Clean the tube under running water and hang it to let it dry completely.

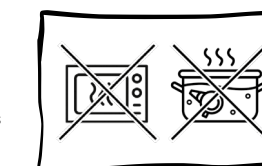
• Let the parts dry completely before the next use.

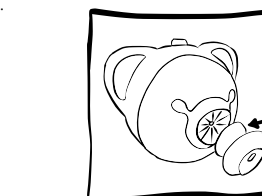• It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts because they might deform by heat!

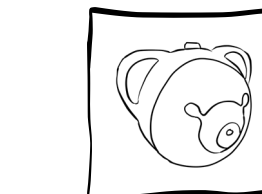
• After use, check the condition of the air mesh. (The air mesh is placed under the connecting unit.)
• The air mesh is there to prevent solid objects and dust from entering the housing.
• The air mesh is not there to absorb any mucus or liquid.
• If the air mesh is dirty or clogged, replace it.
• The air mesh can be used for a very long time next to the proper usage of the device.


• Clean the housing with a dry, soft cloth.
• Do not use any liquids, detergents, or chemicals for cleaning the housing.
• Keep the device in a space where it is protected from dust, direct sunlight, and heat.

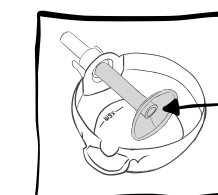HINTS & TIPS

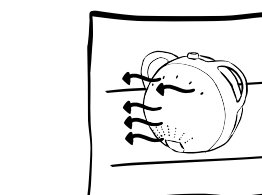• It is forbidden to use the Colibri head and the device without the collared tube.

• While in use, the device should be in a stable position, standing on a flat surface.
• Do not cover the vents on the back of the device while in use. Hot air exits the device through the vents.
• Make sure the heat exits in a different direction than where your child is.

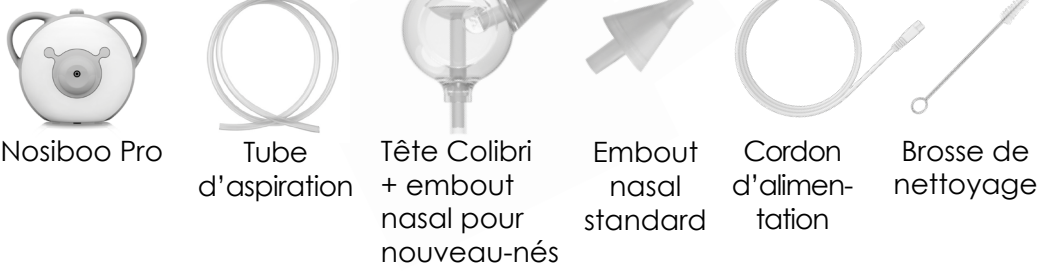
• The device should only be operated by adults.
• Do not allow children to play with the device. It is NOT A TOY.

• Dry, very thick mucus can be dissolved by a mild saline solution. After using it, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.
• Do not use more than 5 ml of saline solution of once.

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