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Rechargeable batteries for Nosiboo devices

Safety Instructions

EN

Danger

1. Do not immerse the battery in liquid such as water, beverages, or other fluids.
2. Do not use or place the battery near fire, heater, or high temperature environment (above 80°C).
3. Do not use unauthorized chargers.
4. Do not attach or insert battery with polarity reversed.
5. Do not connect the battery to an AC outlet or DC automotive plug.
6. Do not use the battery in equipment for which it was not intended.
7. Do not incinerate the battery in fire or heat it.
8. Do not short-circuit the battery by directly connecting the positive and negative terminal with metal object such as wire.
9. Do not apply excessive impact to the battery, such as striking, throwing, trampling, etc.
10. Do not penetrate the battery with a nail or other sharp object.
11. Do not disassemble the battery.

Warning

1. Keep the battery away from small children. If the battery or any of its component parts is swallowed, seek medical attention immediately.
2. Do not place the battery in or near a microwave or other cooking appliances. If subjected to heat or strong electromagnetic radiation, the battery may leak, generate heat, smoke, catch fire, or explode. Do not mix with other batteries. The battery should not be used with other batteries having a different capacity, chemistry, or manufacturer. Doing so could cause the battery to generate heat, smoke, catch fire, or explode.
3. Immediately remove it from the device or charger, and stop using it, if there are noticeable abnormalities, such as smell, heat, discoloration, or deformity. The battery may be defective and could generate heat, smoke, catch fire, or explode with continued use.
4. Do not use a leaking battery near open flame.
5. Do not touch a leaking battery. If liquid leaking from the battery gets into your eyes, immediately flush your eyes with clean water and seek medical attention. If left untreated, it will cause significant eye damage.

Caution

1. Read the manual before use. Keep for future reference.
2. Do not use the battery where static electricity in excess of 1000V is generated as it may damage the protection circuit and cause hidden trouble of safety.
3. Only charge the battery between 0°C and 45°C. Charging outside of this temperature range may cause the battery to leak, generate heat, or result in serious damage. It may also cause the battery's performance and life to deteriorate.
4. Read the charger's manual before use for proper charge method.
5. Please contact the supplier if the battery gives off an unusual odour, generates heat, or shows signs of rust prior to its initial use.
6. Parents must explain how to use the system and the battery. Please check back periodically to ensure children are using the system and the battery correctly.
7. Do not charge or discharge near flammable materials. Doing so could result in fire.
8. If electrolyte leaks from the battery and comes into contact with skin or clothing, immediately flush with water. Otherwise, it may cause skin irritation.
9. If the battery pack have a system interface consisting of stripped lead wires or exposed contact plates, handle with due care. Temporarily insulate exposed contacts and conductors with an insulator such as polypropylene tape or polyvinyl chloride tape. Failure to do so could result in an electrical shock, a short circuit causing the battery to generate heat, smoke, catch fire, or the combustion of other materials.
10. When disposing of the battery, be aware discharged battery may cause fire, tape the terminals to insulate them. Recycle it according to local rules and regulations.
11. In case the battery terminals are dirty, clean the terminals with a dry cloth before use. Otherwise power failure or charge failure may occur due to the poor connection with the instrument.
12. The batteries should be stored at room temperature, charged to about 40% to 60% of capacity. In case of over-discharge, batteries should be charged to 3.90V*n with standard charge method once every 3 months during storage. If batteries are stored for more than one year, they should undergo one standard charge-discharge cycle before use to activate them and restore capacity.

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